



CLAVICLE STRAP

INSTRUCTIONS FOR USE

Single Patient Use Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

- Clavicle fractures
- Postural problems

CONTRAINDICATIONS:

- None

WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CLEANING INSTRUCTIONS:

Unfasten hook and loop straps. Hand wash in warm water using a mild detergent. Air dry only.

WARRANTY:

Manufacturer guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.

PART NUMBERS: 200000-XX

-XX	-01	-03	-05	-07	-08
	XS	SM	MD	LG	XL

LATEX FREE


EUROPEAN REPRESENTATIVE:

CE	EC	REP
	EMERGO EUROPE Molenstraat 15 2513 BH, The Hague The Netherlands Phone: +31.70.345.8570 Fax: +31.70.346.7299	


UNITED SURGICAL 2235 Pennsylvania St. Fort Wayne, IN 46835 USA
--

APPLICATION INSTRUCTIONS:


1. Unfasten hook and loop closure.

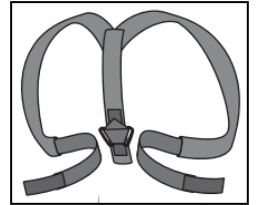


2. Position Clavicle Brace such that the center of the brace is centered on the patient's back. The D-rings should rest furthest from the patient's neck.

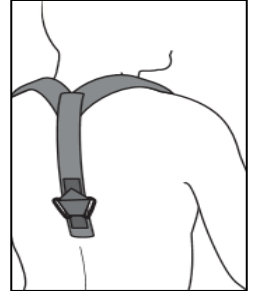


3. Position straps around the top of the patient's shoulders and wrap under the arm pits. Thread the side straps through the D-rings.

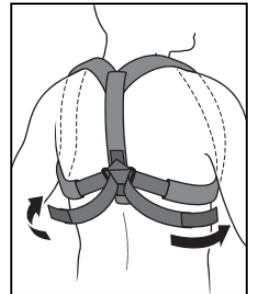

4. Tighten the straps to pull the shoulders back, and fasten the hook tabs on the ends of the straps to the brace.

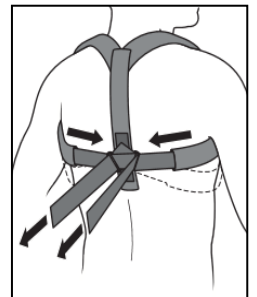

5. The D-ring attachment site can be repositioned on the center strap for additional support and comfort.

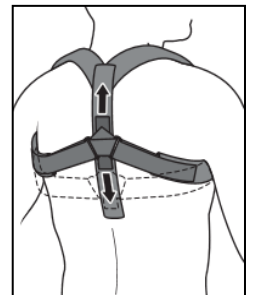

United Ortho Enterprises
 2235 Pennsylvania St. Fort Wayne, IN 46803
 Phone: 800-227-8748; Fax: 877-688-7975
 www.United-Ortho.com